

VelaShape® III

Refining and mastering
body treatments,
maximising ROI

eBook



 **CANDELA™**
Science. Results. Trust.





Dear reader,

On behalf of Candela, we would like to say thank you for your interest in VelaShape® III, our successful device for comfortable, non-invasive body contouring. In this eBook, we provide you with the most important information regarding this device, from technology overview to some of the results clinical experts worldwide have achieved with their patients.

Know that when you decide to work with one or more of our devices, we'll do everything we can to provide you with the highest level of customer service possible.

That's our promise to you.

Candela Marketing Team.

What can you treat?

The VelaShape III system provides a successful solution for comfortable, non-invasive body contouring for a wide range of patients and treatment areas. It's proven to temporarily reduce the appearance of cellulite and thigh circumference and abdomen — areas that women everywhere identify as problem zones — with a deep therapeutic treatment that delivers consistent, reproducible results, with minimal downtime or discomfort.

Non-invasive body shaping treatments

Fat Destruction

Cellulite

Circumference Reduction

Patient advantages

Give your patients what they want:

The assurance of proven results. With over 10 years of clinical experience, more than 5 million independent treatments, and the most published studies of any medical body shaping device, the VelaShape treatment is one of the most recognised non-invasive body shaping treatments on the market today.

Power and speed. The VelaShape III device can achieve end-point temperature in less time, with up to 150 watts of radiofrequency (RF) power. That reduces the time needed for treatment.

Comfort. The VelaShape III procedure is as comfortable as a hot stone massage.

Convenience. The VelaShape III procedure often takes under an hour.

All-season treatment. The VelaShape III device can be used on all skin types in any season and any climate.

Practice advantages

BOOST
YOUR
ROI

Attract new customers.

VelaShape III treatments are affordably priced, enabling you to introduce patients to your practice's body shaping capabilities.

Treatment versatility.

With the VelaShape III device you can treat a broad range of indications, ages, and skin types.

Repeat business.

The VelaShape III regimen keeps patients coming back, and every visit means more exposure to your other aesthetic offerings.

A significant new revenue stream.

Treatments can easily be bundled into packages of four, six, eight, or more.

Easy to use.


The VelaShape III system's operator-friendly design lets you delegate procedures to appropriate medical staff.

A synergistic use of energies


VelaShape III system employs a combination of modalities:

- Infrared light (IR) heats the tissue superficially
- Bi-polar radio frequency (RF) heats tissue up to 20 mm in depth
- Vacuum technology ensures precise delivery of energy
- Mechanical manipulation improves lymphatic drainage and cellulite smoothing.


The combination of IR and vacuum-coupled RF technologies causes deep heating of the fat cells (adipocytes), their surrounding connective fibrous septae and the underlying dermal collagen fibers. Efficient heating of the adipocytes is achieved by higher output of the RF.

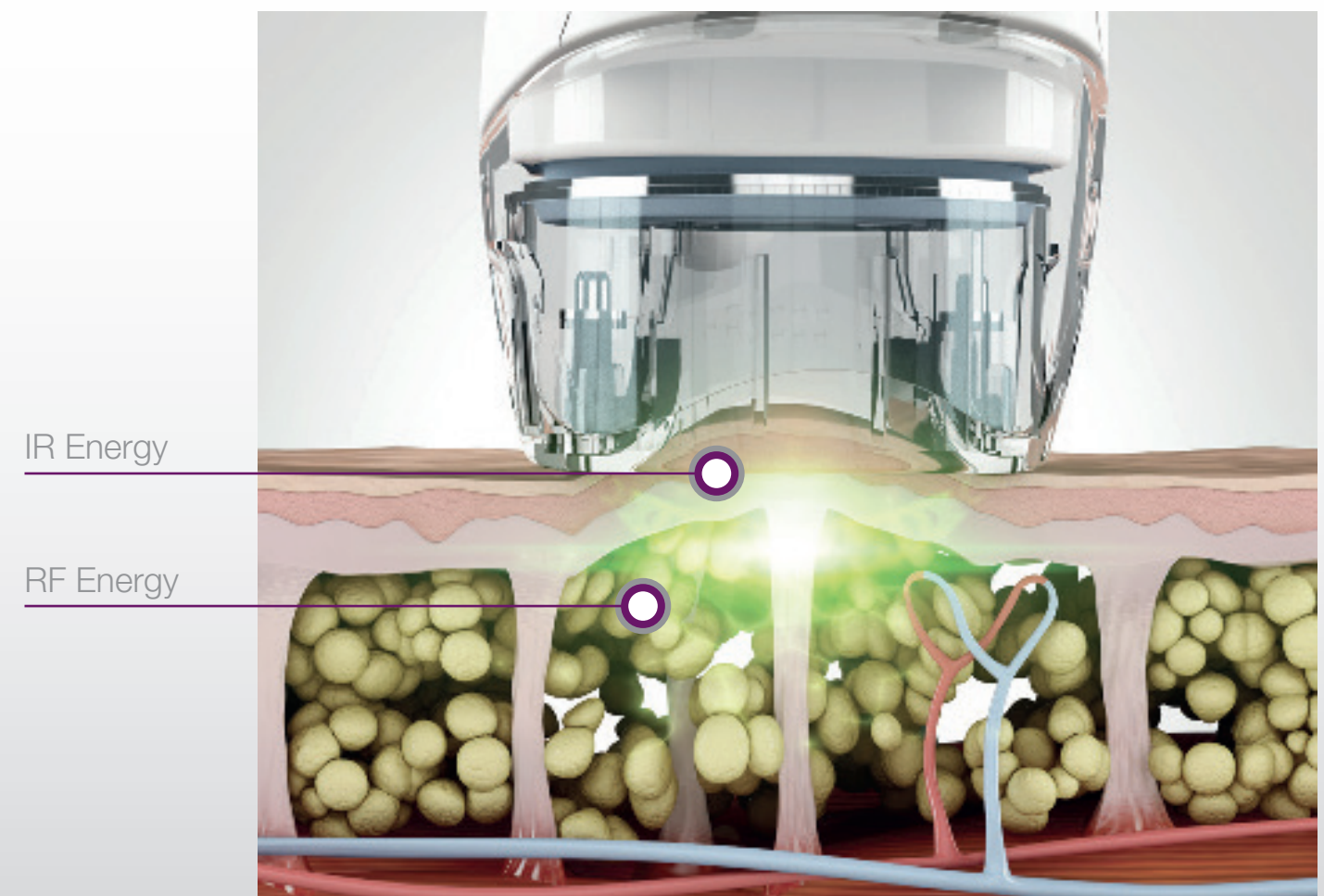


Results | Lifted. Firmed. Sculpted

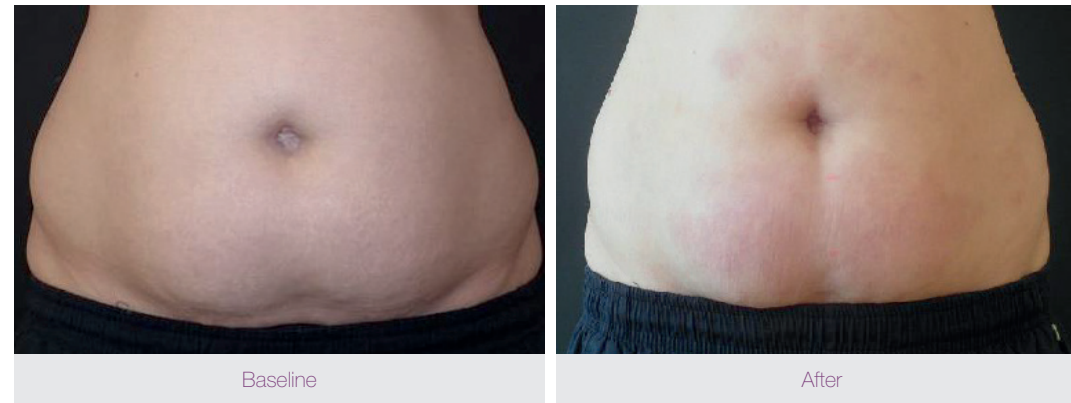

Photos courtesy of Eun Hee Kang, M.D.

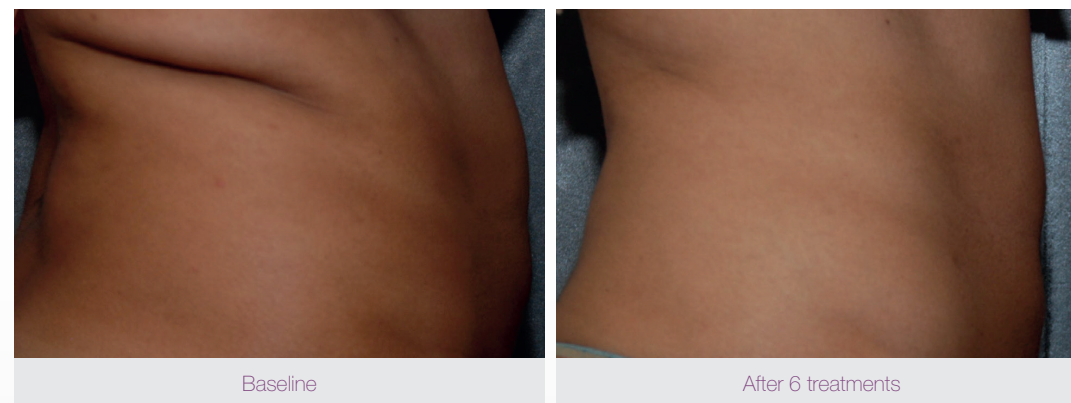


Photos courtesy of Echo V. Rowe, M.D.

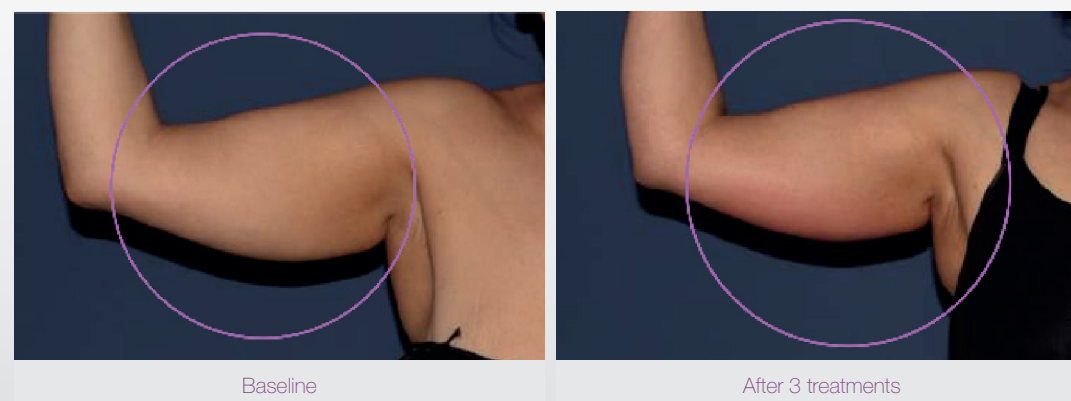

Photos courtesy of Ariel Haus, M.D.

Photos have been unretouched and individual results may vary.

Results | Lifted. Firmed. Sculpted

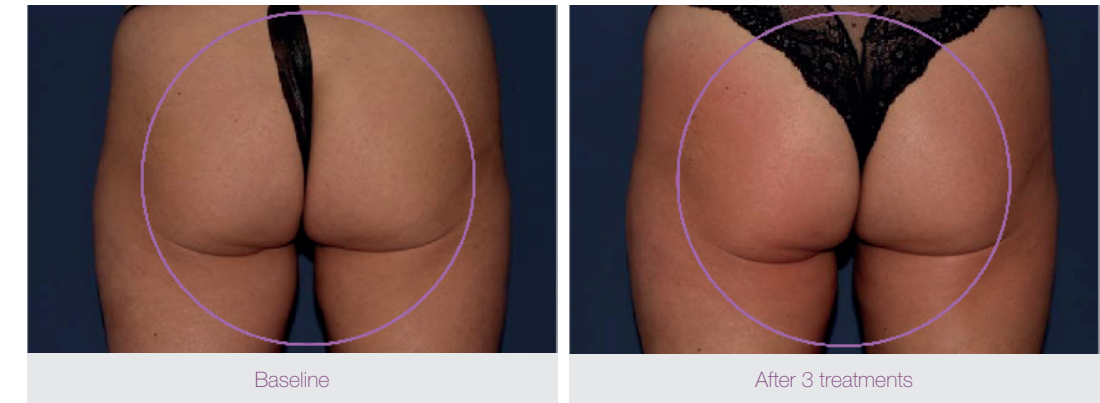

Photos courtesy of Ariel Haus, M.D.




Photos courtesy of Gerald Boey, M.D.



Photos courtesy of Ariel Haus, M.D.

Photos have been unretouched and individual results may vary.


Results | Lifted. Firmed. Sculpted

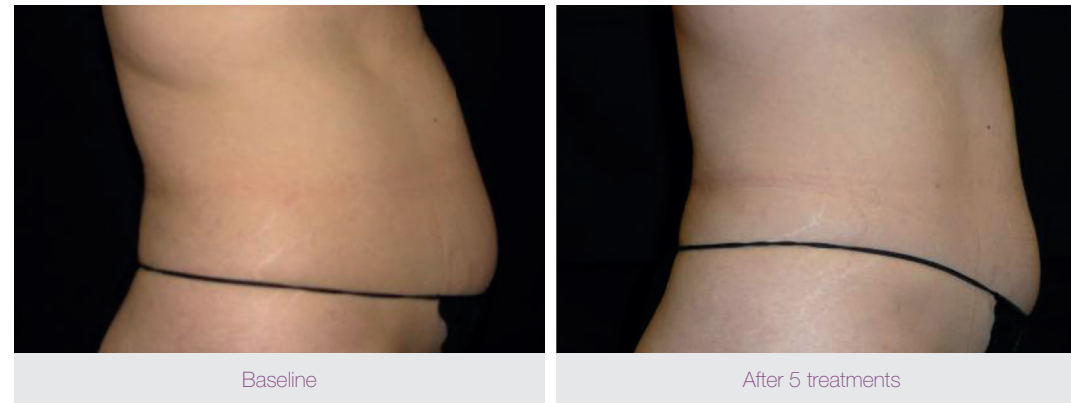

Photos courtesy of M. Winter, M.D.



Photos courtesy of T. Becker, M.D.

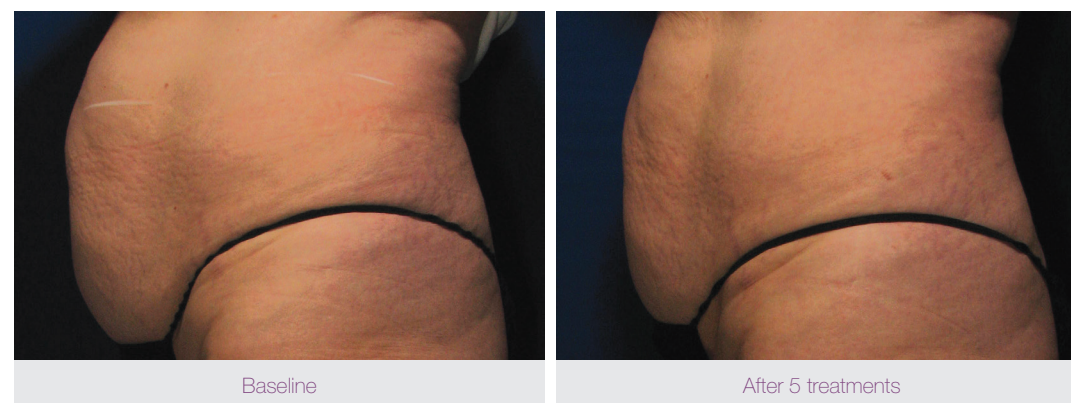

Photos courtesy of Ruthie Amir, M.D.


Results | Lifted. Firmed. Sculpted



Photos courtesy of Ruthie Amir, M.D.



Photos courtesy of Lori Brightman, M.D.

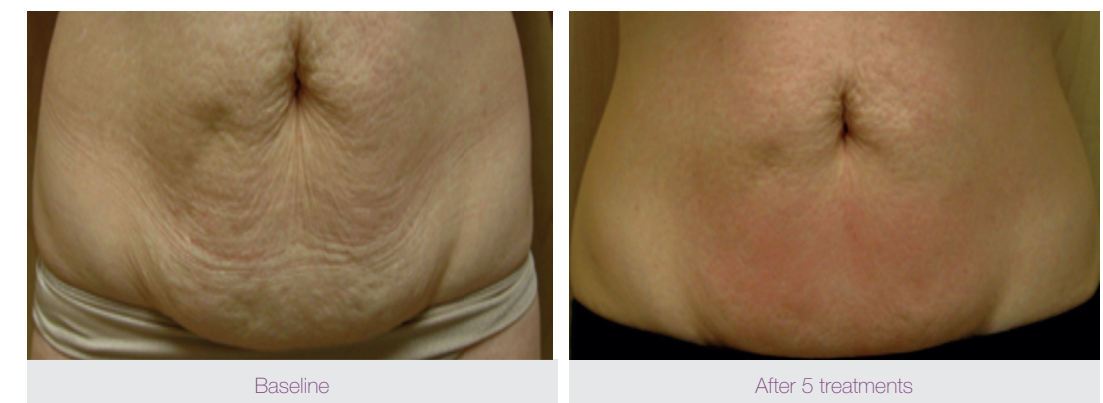

Photos courtesy of Longevity Medical Clinic.

Don't take our word. Take it from our customers.

The most popular body treatment in my clinic is the VelaShape III. Some patients can see improvement just after the first treatment.



Ariel Haus - Dermatologist, M.D.
Dr. Haus Dermatologist Clinic, Harley Street

After trying multiple technologies to treat cellulite and fat deposits, finally I am able to offer my patients a safe and effective treatment that provides visual results from the 1st treatment session. The VelaShape III has proven to be a painless procedure and my patients are able to resume their normal activities immediately after.



Virginia Benítez, M.D.
Humanline Banus Hospital, Spain

We've seen great results consistently with VelaShape. We've had multiple patients lose up to 5 centimetres off their waist and abdomen area.



Lori Brigtmán, M.D.
Board Certified Dermatologist, New York

The VelaShape III system provides the market with a technological evolution in cellulite treatments. I am confident that I can offer my patients a treatment that is going to improve their cellulite. In fact, VelaShape III is a great investment for building patient loyalty, while improving our medical prestige.



Adriana Ribé, M.D.
Ribé Clinic, Spain

Summary of peer-reviewed articles

A prospective clinical study to evaluate the efficacy and safety of cellulite treatment using the combination of optical and RF energies for subcutaneous tissue heating.

Sadick NS, Mulholland RS.

Cellulite treatment using a novel combination radiofrequency, infrared light, and mechanical tissue manipulation device.

Alster TS, Tanzi EL.

Evaluation of the combination of radio frequency, infrared energy and mechanical rollers with suction to improve skin surface irregularities (cellulite) in a limited treatment area.

Kulick M.

Treatment of cellulite with a bipolar radiofrequency, infrared heat, and pulsatile suction device: a pilot study.

Wanitphakdeedecha R, Manuskiatti W.

Post-pregnancy body contouring using a combined radiofrequency, infrared light and tissue manipulation device.

Winter ML

Improvement in Arm and Post-Partum Abdominal and Flank Subcutaneous Fat Deposits and Skin Laxity Using a Bipolar Radiofrequency, Infrared, Vacuum and Mechanical Massage Device.

Brightman L, Weiss E, Chapas AM, Karen J, Hale E Bernstein, L Geronemus R.

Reduction in adipose tissue volume using a new high-power radiofrequency technology combined with infrared light and mechanical manipulation for body contouring.

Adatto, M., et al.

Main conclusions from peer-reviews:



Statistical significance, sustainable reduction in circumference and improvement in appearance of arms and abdomen following treatment.



Significant results in a few short sessions without compromising patient safety or comfort.



It is suggested that the higher bipolar RF energy available in VelaShape III may result in a more intense heating of the targeted tissues, resulting in both faster treatment times as well as improved clinical outcomes.



Well tolerated treatment



High patient satisfaction

AWARDS

Top energy-based treatment pick for achieving aesthetic goals from head to toe in 2019.

Awarded by Dr. Sanjay Grover, plastic surgeon from Newport Beach, California.

and



System Specifications

Radio Frequency Power	Up to 150 W
Infrared LEDs	Up tp 3.2 W
Light Spectrum	850 nm
Vacuum	Pulsed
Weight	44 lbs. / 20 Kg.
Dimensions [W x D x H]	15 x 19 x 52" 38 x 49 x 132 cm
Electrical Requirements	110 VAC; 5A; 50 Hz; Sigle Phase 230 VAC; 2.5A; 50 Hzz; Sigle Phase



Disclaimer: All contents of this material are for informational purposes only and provided by Candela without warranties of any kind. Healthcare professionals are solely responsible for making their own independent evaluation as to the suitability of any product for any particular purpose and in accordance with country specific regulations. The availability of products and the indications mentioned in this material is subject to the regulatory requirements and product registration status in each country. Refer to the User Manual for country specific indications. Products and technical specifications may change without notice. Please contact Candela for more details.

© 2019 Candela Corporation. This material contains registered and unregistered trademarks, trade-names, service marks and brand names of Candela Corporation and its affiliates. All other trademarks are the property of their respective owners. All rights reserved. PU04191EN, Rev. 00

